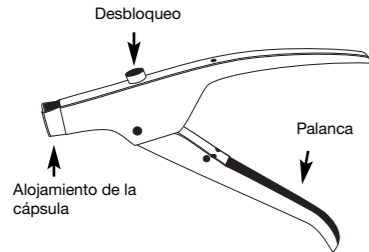
AC Applicator tipo 1

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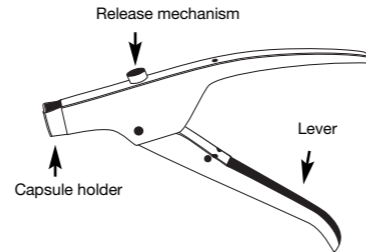
AC Applicator type 1

Instructions for use


Applicator for the direct administration of glass ionomer cement restorative and luting materials in application capsules (AC). The fine aperture on the capsule in combination with the **AC Applicator type 1** allows the material to be precisely applied to the cavity or the restoration for attachment.


Application:

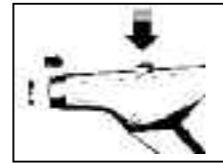
1. Make sure the applicator plunger is retracted/unlocked.

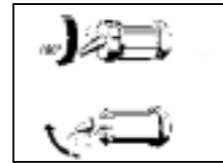
2. Following activation and completion of mixing, open the application capsule in accordance with the manufacturer's instructions.

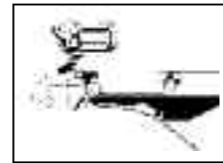3. Insert the application capsule into the applicator, rotate in the desired direction and apply its content into the cavity or the restoration to be attached by slowly and evenly compressing the lever. Do not apply excessive pressure.

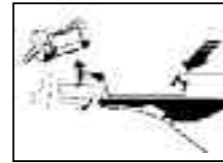4. On completing the application, move the plunger back again by pressing on the release button and lift the application capsule out.

5. Disinfect the applicator after each use. Conventional disinfectant wipes or solutions can be used for disinfection. The **AC Applicator type 1** can be steam sterilised at up to a maximum of 134 °C (steam steriliser, follow manufacturer's instructions).

Additional information:

The **AC Applicator** application system enables you to quickly and hygienically apply the material directly. For detailed information on the different **VOCO** products, please consult the individual instructions for use.

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