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**EN** Instructions for use

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Manufacturer | Hersteller  
Distribution | Vertrieb

Distribution | Vertrieb D/A

**Amann Girrbach AG**  
Herrschaftswiesen 1  
6842 Koblach | Austria  
Fon +43 5523 62333-105  
Fax +43 5523 62333-5119  
austria@amanngirrbach.com  
www.amanngirrbach.com

**Amann Girrbach GmbH**  
Dürrenweg 40  
75177 Pforzheim | Germany  
Fon +49 7231 957-100  
Fax +49 7231 957-159  
germany@amanngirrbach.com  
www.amanngirrbach.com

Made in the European Union

32195-FB 2016-08-08

DE

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Original-Gebrauchsanweisung

### PRODUKTBESCHREIBUNG

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EN

## INSTRUCTIONS FOR USE

Translation of the original instructions for use

### PRODUCT DESCRIPTION

Ceramill Liquid Eye code colours are used for staining the virtually colourless Ceramill Liquid before application to the zirconia restoration. This facilitates handling when immersing or brushing zirconia restorations.

Ceramill Liquid Eye are liquids with water-based organic colouring agents and burn out without residue during sintering. The code colours are available in blue, red, green and yellow.

### APPLICATION

Add Ceramill Liquid Eye to the Ceramill Liquids in the following quantities.

Ceramill Liquid Eye (drops)	Ceramill Liquid (ml)
1	2
5	10
25	50

#### Note:

- \_ If darker or lighter colouring is required, add more or fewer drops of the code colour.
- \_ The intensity of the code colours can vary depending on the material.
- \_ Different colours can be applied overlapping, which produces mixed colours.
- \_ Avoid applying code colours blue and green next to each other, as the two colours are very similar.
- \_ Repeated application of the red code colour increases the intensity of the colour. It is therefore recommended to use this code colour in the body region with monolithic crowns. In this way the shade gradient can be made visible using the code colour.

### RECOMMENDED SERVICE LIFE

Approx. 2 years.

### PACKAGING

760480	Ceramill Liquid Eye Complete Set	
Contents:	760481	Ceramill Liquid Eye red (25ml)
	760482	Ceramill Liquid Eye green (25ml)
	760483	Ceramill Liquid Eye blue (25ml)
	760484	Ceramill Liquid Eye yellow (25ml)

### WARRANTY

Recommendations on technique, regardless of whether are given orally, in writing or as part of practical instructions, are based on our experience and tests and should only be regarded as guidelines. Our products undergo continuous development. We therefore reserve the right to make changes in the design and composition.

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