FOR PROFESSIONAL DENTAL USE IN RECOMMENDED INDICATIONS ONLY (SEE 2. INDICATIONS FOR USE).

Product description

Ceramir® Crown & Bridge is a permanent, radiopaque, bioceramic luting agent. Ceramir® Crown & Bridge is a composite of ceramic and glass in the liquid, water, the cement, which hardens as a result of a chemical reaction, is self-setting.

In combination with Ceramir® QuikCap, the cement is supplied in a capsule for mixing and clinical application. The minimum volume of mixed cement delivered from one capsule is 0.17 ml.

A capsule applicator and a mixing device are required for use. We recommend Ceramir Applicator and Ceramir Applicator 2 from Doxa Dental AB.

2. Indications for use

Ceramir Crown & Bridge is intended for permanent cementation of:

- Metal and porcelain teeth
- Gold and/or porcelain crowns
- Cast or prefabricated metal posts
- High-strength ceramic crowns and bridges suitable for conventional cementation (e.g. zirconia, alumina, and lithium-disilicate)

Contraindications

• Do not use as a direct pulp-capping agent. In case of close proximity to the pulp, use accepted standard clinical procedures.

Precautions

Ceramir Crown & Bridge contains polyacrylic acid, which may be irritant to eyes and skin. Avoid contact with eyes. In case of skin contact, rinse with plenty of water. In case of contact with eyes, rinse immediately with water and seek medical advice.

• Do not use on patients with known allergy to polyacrylic acid. In rare cases, the product may cause sensitivity to some persons. If such reactions are observed, discontinue the use of the product and consult a physician.

3. Step-by-step instructions

Note: Do not use any pre-treatment agents, e.g. slanos, primers or bonding agents, when cementing with Ceramir Crown & Bridge.

Use of any such pre-treatment may compromise the bonding of the cement.

3.1 Tool preparation

Each with HF according to manufacturer's instructions. Do not use any pre-treatment agents, e.g. slanos, primers or bonding agents.

All other restorations

Do not use any pre-treatment agents, e.g. slanos, primers or bonding agents when cementing with Ceramir Crown & Bridge. Use of any such pre-treatment may compromise the bond between cement and restoration.

Ab orusion can be recommended by the restoration manufacturer.

3.3 Pulp protection

• If pulp protection is required, use accepted standard clinical procedures.

3.4 Activation

ATTENTION: WORK IMMEDIATELY Starting immediately following activation (the reaction between the liquid and powder starts directly after activation).

• Open the aluminum foil and remove the capsule.

• Hold the capsule firmly against a hard, flat surface with the plunger facing down (fig. 1).

• Push down firmly, forcing the plunger into the body of the capsule (up to the rim) (fig. 1).

3.5 Mixing

• Immediately place the capsule in a high-frequency oscillating (4.000 to 5.000 rpm) or rotating capsule mixer (fig. 2).

• Mix for 8 seconds.

3.6 Application

• Immediately mix the capsule into the applicator using the grooves to the right of the capsule (fig. 3a).

• Gently close the dosing lever (fig. 3a) two times to push the cement towards the tip.

• Apply the cement by gently squeezing the lever two more times (fig. 3).

• After application, remove the empty capsule using the applicator release button (fig. 3b).

Working and setting time

• Working time (from the start of mixing) at 23°C/73°F: 2 minutes.

• Setting time (from the start of mixing): 3 to 8 minutes.

Note: Room temperature above 23°C/73°F will accelerate setting and reduce the working time available.

3.7 Sealing and finishing

• Seal the restoration and stabilize it under pressure, to avoid any movements, until the cement reaches a rubber-like consistency (approximately 30 minutes).

• Remove excess material when the cement has reached the rubber-like consistency (fig. 8).

• Keep the restoration stable for an additional 4 minutes (fig. 9).

• Clean the margins and the margins of the restoration (especially the gingival side) and remove any remaining cement.

• Advise the patient not to load their restoration for the first hour after cementation.

4. Cleaning the applicator

• Ceramir Applicator can be cleaned with standard cleaning agents, e.g. alcohol-based spray disinfectants.

5. Storage and shelf life

• Keep Ceramir Crown & Bridge QuikCap capsules at temperatures between a 4°C/39°F and 20°C/68°F. Do not open the aluminum pouch until immediately before use.

• Do not store Ceramir Crown & Bridge QuikCap capsules after their expiry date.

Warning

If this product is found defective within the Warranty Period, your exclusive remedy and Doxa Dental AB's sole obligation and liability shall be the repair or replacement of the defective product. DOXA DENTAL AB IS NOT LIABLE FOR ANY LOSS OR DAMAGE ARISING FROM THE PRODUCT, WHETHER DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL, CONSEQUENTIAL, INCLUDING LOSS OF PROFITS, REGARDLESS OF THE THEORY ASSERTED, INCLUDING WARRANTY, BREACH OF CONTRACT, NEGLIGENCE OR STRICT LIABILITY.

Trademarks

Ceramir® is a trademark of Doxa Dental AB.

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