VOCO**Dimanto®****Instructions for use****Product description:**

Dimanto is a one-step polishing system for pre-polishing and high-gloss polishing finished composite restorations. The diamond-interspersed silicone polishers are suitable for repeated use.

Method of use:

The **Dimanto** polishers must be inserted as deeply and firmly as possible into the contra-angle handpiece. The polisher should have reached the desired rotational speed before making contact with the composite. Avoid jamming or levering during polishing, since this will lead to an increased risk of fracture. Polishers that are bent or no longer smooth-running must be promptly sorted out and discarded. Safety glasses should always be worn when working with **Dimanto** polishers. Supports, shanks or the object being polished may break due to improper use or material defect. Alternatively, the work can be carried out behind a protective panel of glass. A face mask must be worn to avoid inhalation of dust from the polisher or the restoration. Dust extraction is also advisable.

Rotational speed instructions:

Compliance with the recommended rpm range of 3,000 min⁻¹ - 8,000 min⁻¹ leads to best work results. The maximum acceptable rpm (20,000 min⁻¹) must never be exceeded. **Dimanto** polishers can be used with or without water spray cooling. Care should be taken (minimal feeding pressure, reduced rpm) to avoid generating excessive heat, especially when not using water cooling. Irreversible damage to the tooth and the surrounding tissue can be the result of excessive heat generation.

Pressure:

Work with moderate feeding pressure when beginning to polish. Work with reduced pressure during the final stage of polishing to achieve maximum gloss. Always avoid excessive pressure, as it causes temperatures that can lead to pulp damage. In extreme cases, excessive pressure may break the instrument, which can cause injuries.

Hygiene:

All instruments are delivered unsterile and must be run through the indicated cycle before and after each use.

Preparation and processing instructions**1. Cleaning preparation**

Pre-clean under running water with a brush (plastic) directly after use.

2a. Cleaning, manual

Under running water with a brush (plastic bristles).

Machine assisted:

- Ultrasonic assisted cleaning with a suitable cleaning agent and disinfectant.
- Rinse the instruments afterwards under running water.

or**2b. Cleaning, automatic**

Thermal disinfectant: Manufacturer's specification according to DIN EN ISO 15883.

Cleaning programme as indicated by the manufacturer in the operating instructions.

3. Disinfection

Use a solution classified as suitable for **rubber and silicone polishers and synthetics** by the disinfectant manufacturer. Exposure times and concentrations recommended by the manufacturer should be adhered to.

4. Rinsing

Rinse the polishers with distilled water after treatment with cleaning and disinfectant solutions.

5. Drying

Dry with fresh, clean, lint-free cellulose tissues.

6. Maintenance

Visual check of all instruments with optical magnification (5 - 10 fold).

7. Inspection

No residues → continue to **sterilisation**.

Visible residues → repeat **cleaning**.

Reject and dispose of instruments in the event of discernible defects.

8. Sterilisation

Steam sterilisation: Appliance according to EN 13060, validated procedure. Category: S- or B-steriliser.

- Holding time: Full cycle 5 min

- Sterilisation temperature: 134 °C

- Drying time: 10 min

- Threshold values of contents for feed-water and steam condensates

- Loading of steriliser according to manufacturer's instructions

- Follow manufacturer's operating instructions

9. Storage

Store instruments packed and protected from recontamination in proven suitable sterile packaging, cassettes or retainers.

Additional information:

Repetitive reprocessing can change both the look and feel of the instruments slightly, but does not interfere with the instrument's function.

Warning notices:

Strong acids and strong bases may oxidize the stainless steel shaft.

Ultrasonic bath must not exceed temperatures of 42 °C because of the possible coagulation of albumen.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2380 E1 0917 99 © by VOCO
VOCO GmbH