**VOCO****Grandio® Flow** CE 0482

in accordance with EN 24049/ISO 4049

Instructions for use

**Grandio® Flow** is a light-curing flowable restorative combining proven composite technology with innovative nano-technology. The radiopaque Grandio® Flow contains 80% w/w inorganic fillers (= 65.6 % vol.) in a methacrylate matrix (Bis-GMA, TEGDMA, HEDMA) and cures under blue light (halogen/LED). It is suited for restorative therapy in anterior and posterior areas. Because of its low viscosity, the material exhibits very good wetting and adaptation to cavity walls. **Grandio® Flow** is used with a dentine-enamel bond.

**Shades:**

A1, A2, A3, A3.5, A4, B1, B2, C2, D2, OA2, BL, WO

**Indications:**

- filling minimally invasive cavities of all classes
- filling small class I cavities and extended fissure sealing
- filling class II - V cavities including V-shaped defects and cervical caries
- blocking out undercuts
- lining or coating cavities
- repairing fillings and veneers
- luting translucent prosthetic pieces (e.g. full ceramic crowns, etc.)

**Application:**

**Grandio® Flow** is a material which adapts very well to the cavity, helps to avoid air enclosures and accelerates restorations. Prevent the flowing-off of **Grandio® Flow** in critical areas (e.g. wedge a transparent matrix in approximal areas).

**Preparation/Shade selection:**

Clean the teeth to be treated with a fluoride-free cleaning paste. Mark occlusal contact points (posterior area); a minimal separation facilitates the design of the approximal contact and the placement of the matrix. To select the shade, compare the **Grandio®** shade-guide with the cleaned tooth before anaesthesia (moisten shade-guide and tooth, select shade in daylight).

**Drying:**

Provide a sufficiently dry workflow. Using a rubber dam is recommended.

**Cavity preparation:**

Cavity preparation should generally be conducted according to the rules of adhesive filling therapy, e.g. minimally invasive to save healthy tooth substance. Bevel enamel margins. Observe special preparation forms in extended fissure sealing and deciduous teeth! Then clean and dry the cavity. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

**Lining:** In areas close to the pulp, a calcium hydroxide lining should be applied. Cover **Calicur** with a stable cement.

**Bond:**

**Grandio® Flow** is used in the adhesive technique with a dentine-enamel bond. All light-curing bonding materials can be used. Follow the respective instructions for use with regards to preparation (etch technique) and application.

**Application of Grandio® Flow:**

Let the material reach room temperature before application. Place a matrix on cavities in the approximal area. Using transparent matrices is advantageous.

Lock the application cannula (type 41) onto the syringe with a clockwise turn and apply **Grandio® Flow** directly. For fillings of more than 2 mm, apply and polymerise in layers. In order to ensure the function of the non-dripping syringe do not pull back the plunger during or after use.

Insert Caps into the opening of the dispenser following the respective instructions for use. Turn Caps to the desired direction and remove the protective cover. Apply **Grandio® Flow** directly into the cavity by slowly and evenly pressing the levers of the handle together.

**Light-curing:**

Conventional polymerisation devices are suitable for light-curing this material. Halogen light devices should have a minimum light output of 500 mW/cm<sup>2</sup>. LED lamps should have a minimum output of 300 mW/cm<sup>2</sup>. The curing time is 20 s minimum per layer, for opaque shades A4, OA2 and WO a minimum of 40 s. Hold the tip of the curing light as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth will be reduced. Insufficient curing can lead to discoloration and pulp irritation.

**Finishing:**

The filling can be finished and polished immediately after removing the shaping aids (e.g. fine and extra-fine grain diamonds, polishing discs). During finishing and polishing, the filling must be cooled. The margin of the filling, or even better, the entire tooth should be fluoridated as a final step.

**Warnings/Precautions:**

- No known side effects. Hypersensitive patients may experience sensitivity.
- If the prescribed application technique cannot be used, or in cases of known allergies to these ingredients (Bis-GMA, HEDMA, TEGDMA, BHT) **Grandio® Flow** may not be used.
- The enclosed application cannulae are for one use only and must be discarded after application. Close the syringes after use with the provided cap.
- Phenolic substances, especially eugenol and thymol-containing preparations, disturb the curing of filling composites. Do not use zinc oxide-eugenol cements or other eugenol-containing substances with filling composites.

**Storage:**

Store at temperatures between 4°C - 23°C. **Grandio® Flow** may be refrigerated, but must be allowed to reach room temperature before use. To avoid exposure to light and thus premature polymerisation, syringes should be closed immediately after dispensing. Do not use **Grandio® Flow** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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**Grandio® Flow** conventional polymerisation devices are suitable for light-curing this material. Halogen light devices should have a minimum light output of 500 mW/cm<sup>2</sup>, LED lamps should have a minimum output of 300 mW/cm<sup>2</sup>. The curing time is 20 s minimum per layer, for opaque shades A4, OA2 and WO a minimum of 40 s. Hold the tip of the curing light as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth will be reduced. Insufficient curing can lead to discoloration and pulp irritation.

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**Light-curing:**

Conventional polymerisation devices are suitable for light-curing this material. Halogen light devices should have a minimum light output of 500 mW/cm<sup>2</sup>, LED lamps should have a minimum output of 300 mW/cm<sup>2</sup>. The curing time is 20 s minimum per layer, for opaque shades A4, OA2 and WO a minimum of 40 s. Hold the tip of the curing light as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth will be reduced. Insufficient curing can lead to discoloration and pulp irritation.

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