Harmonize: Nanohybrid Universal Composite

DESCRIPTION

nonize is a light-cured, esthetic, resin-based nanohybrid dental restorative designed for the placement of direct restorations. Harmonize is designed to be used after placement of a dental adhesive, such as a product from within the OptiBond family. Harmonize is available in Unidose composite compules and syringe delivery systems.

FNGLISH

INDICATIONS FOR USE

Harmonize, a nanohybrid dental restorative, is indicated for direct placement in all cavity classes in anterior and posterior teeth. Additional indications include: repair of enamel defects, repair of provisionals, repair of porcelain restorations, minor occlusal build-ups core build-ups, and incisal abrasio

CONTRANDICATIONS

WARNINGS

- Do not use this product on patients with known methacrylate allergies. Uncured methacrylate resin may cause contact dermatitis and damage to the pulp.
- Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water and soan
- Get medical attention for eves.

PRECAUTIONS

Kerr recommends the following precautions when using Harmonize or any other

- Provide the patient with protective evewear
- Use a rubber dam to isolate the oral cavity (Kerr recommends Kerr OptiDam™ Avoid fluid contamination during placement
- Use of protective gloves and a no-touch dinical technique is strongly recommended If the composite is touched with commonly used gloves, acrylate may penetrate the
- glove. Dispose of glove immediately and wash hands.

The Unidose tips are designed for SINGLE PATIENT USE ONLY to prevent cross-contamina-

ADVERSE REACTIONS

INSTRUCTIONS FOR USE

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- interfere with the shade selection process
- Select the desired shade. A composite mock-up or VITA-matched shade guide may be used to check the color of the tooth to be restored and any adjacent teeth. In many cases, a single shade of Harmonize in enamel onacity will be sufficient to restore composite. Shade selection and mock-ups should match the thickness of the tooth
- After shade selection, isolate the tooth with your preferred isolation method. Prepare the tooth using your preferred rotary instruments and using contemporary cavity designs. Harmonize offers excellent blending (chameleon effect) which may
- be further enhanced by the use of slight bevels on preparation margins. If replacing a restoration, be sure to remove all existing restorative material from the operative site If a pulp exposure has occurred, place a small amount of calcium hydroxide (Ker recommends the use of Life) followed by the placement of a conventional glass
- EC REP Kerr Italia S.r.l. Via Passanti, 332, I-84018 Complete the honding procedure. Harmonize is compatible with all methacrylate dental adhesive systems. OptiBond products have been shown to produce the rongest and most durable restorative interfaces.
 - red, place a liner with a low-viscosity flowable composite, such as Herculite™ XRV Ultra Flow, and light cure according to manufacturer's instructions.
 - Harmonize features optimal, non-sticky handling. It permits the use of hand nts to define most of the restoration anatomy, which wi**ll** he**l**p minimiz the time and effort associated with rotary instrumentation. For Unidose delivery: Inject composite material from the Unidose tip directly into the cavity, starting with the deepest portion. For Syringe delivery: After extruding material onto pad, back off syringe at least two full turns to relieve pressure on the material.
 - Apply composite layers up to 2mm in height for enamel/translucent (incisal) shades or 1.5mm in height for dentin shades, taking care to properly adapt the material to A guide for using the cavity surface.
 - 10. Cure each layer. Refer to the table at the end of these instructions for recommended curing times of the various shades at various light intensities. Avoid exposure of rmonize to intense operatory light.
 - naminuze to intense operatory tight.

 After curring, complete the finishing and polishing procedure. Adjust occlusion as needed. Kerr recommends Kerr Trimmling and Finishing Carbides, such as the TDF™ (Trim Define Finish) and OA™ (Occlusal Anatomy) series burs, for finishing and adjustments and Kerr Opti 1Step Polisher one-step, multi-use polisher for polishing. Kerr Opti 1Step Polisher has been found to produce equivalent polish to other multi-step polishers tested in half the time.

mends additional 10 second cures on both the buccal and lingual surfaces of the tootl

*Recommended Curing Times

Adequate curing is essential for restorative function. Uncured or undercured composite material is far more likely to experience some form of clinical failure. Kerr recommends Manufactured By: Kerr Corporation regular inspections of your curing lights to ensure they are functioning and providing 17 West Collins Avenue Orange, CA 92867 USA the expected irradiance. Harmonize is intended to be cured by a halogen or LED light with a minimum light intensity of 600 mW/cm2 and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations. All 1-800-KERR-123 / www.kerrdental.c EC REP Kerr Italia S.r.l. Via Passanti, 332, I-84018 Scafati (SA) Italy | +39-081-850-8311 | kerrdental.eu times refer to a single occlusal surface cure. In any posterior restoration, Kerr recom

Rx ONLY	DMC2944_Rev00		Dentin Shades	Enamel/Transluc (Incisal) Shade
		Incremental Depth	1.5 mm	2.0 mm
		Optilux (or light with output 600 — 1000 mW/cm²)	2	0 Seconds
		Demi Ultra / Demi Plus (or light with output >1000 mW/cm²)	1	0 Seconds

Disinfection and Storage

Syringes may be safely disinfected. Use an intermediate level disinfectant, such as Cavicide nd Caviwipes, or as recommended by the Centers for Disease Control. Refer to manufac-irer's instructions for recommended time, dilution, and solution temperature. Harmonize should be stored at ambient temperature. Avoid excess heat. If product is stored in a refrigerator allow the product to reach room temperature prior to use. Every effort should be made to protect the composite from visible light prior to use.

The expiration date is marked on packaging. Do not use restorative after expiration date. For proper disposal always follow Federal or State lay

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

propose for the vertical position, but the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product. In order to take advantage

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Tonalità amatto/
tradicionato/
tradiciona of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr he liable for any indirect incidental or consequential damages. EXCEPT AS EXPRESSIN PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED. INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR

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