# SpofaDental

A Kerr Company

 **SpofaDental a.s.**, Markova 238, 506 01 Jičín, Czech Republic  
+ 420 493 583 111, www.spofadental.com. Made in Czech Republic

## Isodent<sup>®</sup>

en

### Gypsum separating fluid

**Characteristic:** Isodent is an alginate solution designed for insulating plaster moulds. Its main advantage is its insulating ability also on not entirely grease free plaster mould surface.

**Indications:** Pour the needed amount of Isodent to a small container and spread with a flat brush over the still warm plaster mould (approx. 40°C). Apply in a thin layer. If repeating is necessary, repeat after the first layer gets dry.

**Danger:** Hazardous ingredients : formaldehyde ...%. H317 - May cause an allergic skin reaction. H350 - May cause cancer. P201 - Obtain special instructions before use. P202 - Do not handle until all safety precautions have been read and understood. P261 - Avoid breathing spray, vapours, mist. P272 - Contaminated work clothing should not be allowed out of the workplace. P280 - Wear eye protection, protective gloves, protective clothing. P302+P352 - IF ON SKIN: Wash with plenty of soap and water. P308+P313 - IF exposed or concerned: Get medical advice/attention. P333+P313 - If skin irritation or rash occurs: Get medical advice/attention. P362+P364 - Take off contaminated clothing and wash it before reuse. P501 - Dispose of contents/container to a hazardous or special waste collection point.

#### Emergency telephone number:

United Kingdom: National Poisons Information Service (Newcastle Unit), Claremont Place Newcastle-upon-Tyne, Newcastle: +44 191 2606182/+44 191 2606180 24H

**Waste material disposal:** The residue of unused product and contaminated packaging material should be transferred to a company authorized to dispose of waste materials. Observe your local legal regulations.

**Storage:** Store in a dry and dark place at 5 °C to 25 °C, in a well sealed container.

**Note:** Do not expose the product to temperatures under freezing point. At lower temperatures, the preparation could be devoluted.

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