SpofaDental

A Kerr Company

 SpofaDental a.s., Markova 238, 506 01 Jičín, Czech Republic
+ 420 493 583 111, www.spofadental.com.

Ceramic casting ring Liner KP-50

en

Casting ring liner

Characteristic: Liner made from ceramic refractory fibres, intended for lining of metal casting rings before they are filled with casting investment material. The product does not contain asbestos.

Application: Easily compressible liner allows optimum setting expansion as well as thermal expansion of all investment materials. In this way it perfectly eliminates thermal contraction of the alloy being cast.

Processing: Line the casting ring with undamped liner, leaving free margin of approximately 3 mm at both ends of the ring.

Waste Disposal: The residue of unused product and contaminated packaging material should be transferred to a company authorized to dispose of waste materials. Observe your local legal regulations.

Storage: Store in a dry and clean well-ventilated place at 5-25 °C, in a well sealed container.

Shelf life: Unlimited.

Package: 20 m of tape 50 mm wide

Danger: Hazardous ingredients : Aluminosilicate wool - amorphous. H315 - Causes skin irritation. H350 - May cause cancer. P201 - Obtain special instructions before use. P264 - Wash hands thoroughly after handling. P280 - Wear eye protection, protective gloves, protective clothing. P302+P352 - IF ON SKIN: Wash with plenty of soap and water. P308+P313 - IF exposed or concerned: Get medical advice/attention. P332+P313 - If skin irritation occurs: Get medical advice/attention. P362+P364 - Take off contaminated clothing and wash it before reuse. P501 - Dispose of contents/container to a hazardous or special waste collection point.

Emergency telephone number:

United Kingdom: National Poisons Information Service (Newcastle Unit), Claremont Place Newcastle-upon-Tyne, Newcastle: +44 191 2606182/+44 191 2606180 24H

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