Repin™

Zinc Oxide-Eugenol Impression Material

Indications:

- Impressions of toothless jaws in individual impression trays for making removable full dentures.
- Temporary fixations of crowns, bridges and pin teeth.

Contraindications:

Not described.

Undesirable effects:

Presence of eugenol in the product may cause allergic reactions of very sensitive people.

Mixing ratio:

In volume: 1 : 1

Application:

Both the pastes are mixed together on a grinding support in the volume ratio 1:1 in the optimum amount according to the size of the impression. The perfectly mixed impression substance is placed into a previously-made individual impression tray. After this tray is inserted, the patient makes a few functional movements with his mouth according to the dentist's instructions. The impression is taken out of the patient's mouth after about 2 minutes and cast with dental stone. For a perfect impression of soft rugae, framing of individual impression trays with thermoplastic impression substance DENTIPLAST is recommended.

Warning:

White paste: H410 Very toxic to aquatic life with long lasting effects. P273 Avoid release to the environment. Brown paste: H317 May cause an allergic skin reaction. P262 Do not get in eyes, on skin, or on clothing. P270 Do not eat, drink or smoke when using this product.

Waste material disposal:

Unused material and the packaging should be transferred to a company authorized to dispose of waste materials or dispose of it in an incineration plant for dangerous waste.

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