



ceram.x[®] SphereTEC[™] one

universal nano-ceramic restorative

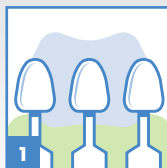


Shade selection

Use the VITA[®]* Shade Guide and i-shade label or the ceram.x[®] SphereTEC[™] one shade fingers.

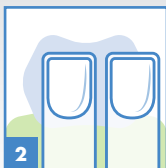

Conditioning and/or bonding

Apply acid conditioner followed by adhesive or self-etching adhesive. Consult Instructions for Use.

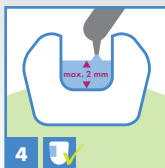


Placement of restorative

Place increments of a maximum of 2 mm.



Light-curing

Light-cure increments.
* $\geq 500 \text{ mW/cm}^2$: 20 sec
 $\geq 800 \text{ mW/cm}^2$: 10 sec
(e.g. with SmartLite[®] PS or SmartLite[®] Focus[®])

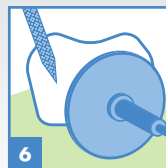

Finishing and polishing

Shape and contour, finish and polish (e.g. with Enhance[®] and PoGo[®]).

K101600-00

DENTSPLY

Farben: Cyan Magenta Gelb Schwarz

K101600-00 ceram.x SphereTEC one_ITG_04.indd 1

Format: 80 x 150 mm breit

12.11.2015 16:22:45

ceram.x[®] SphereTEC[™] one

universal nano-ceramic restorative


Shades available

ceram.x[®] SphereTEC[™] one CLOUD shades:
A1, A2, A3, A3.5, A4

Instructions

Shade	 mW/cm ²	 sec	 mm
A1 - A4	≥ 500	20	2
	≥ 800	10	

i-shade label

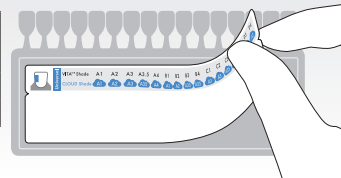

i-shade |  Stick i-shade label to VITA[®]* Shade Guide

Caution, consult
instructions for use

Shade Guide



VITA[®]* Shade

CLOUD Shade

A1 B1 C1

A2 B2 D2

A3 C2 D3 D4

A3.5 B3 B4 C3

A4 C4



* VITA[®] is not a registered trademark of DENTSPLY International Inc.

© DENTSPLY DeTrey 2015
www.dentsply.eu