


DENTSPLY

# Spectrum<sup>®</sup>TPH<sup>®</sup>3

Universal microhybrid composite restorative

Preparations:

- Clean tooth with rubber cup and cleaning paste
- Select shade and opacity
- Isolate (rubber dam)
- Clean freshly cut surfaces with water spray
- Air-dry

 mW/cm <sup>2</sup> ≥ 500	Shade	mm	sec
	A1, A2, A3, A3.5, B1, I-B1, B2, C2, C3, D3	3	20
	O-A2, O-A3.5, A4, B3, C4	2	



Caution, consult Instructions for use



Manufactured by:  
DENTSPLY DE TREY GmbH  
78467 Konstanz, GERMANY



© DENTSPLY DE TREY 2013-04-23  
www.dentsply.eu  
www.dentsply.com

Farben: Cyan Magenta Gelb Schwarz


## Summary of application in the Total Etch Technique

Etch




Enamel and dentine pre-treatment

Start at enamel

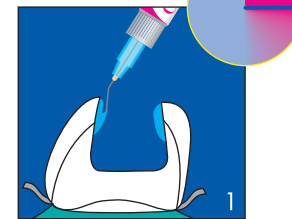

Extend to dentine

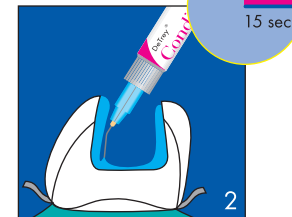

Remove gel and excess water



Do not desiccate


Prime and Bond

Nano-Technology Dental Adhesive




One Layer


Apply ample amounts, leave undisturbed

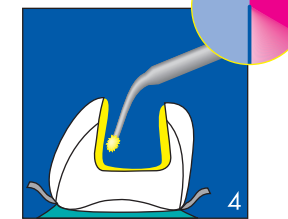

Remove solvent with air-syringe

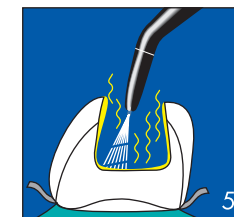

Light-cure

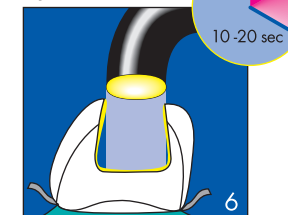


Restore




Application with CompuLes<sup>®</sup> tips


Place increments

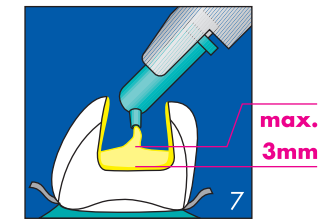

Light-cure each increment

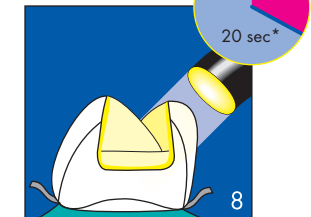

Polishing can be done immediately

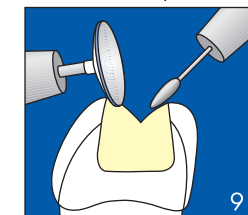

\* Curing Light Output ≥ 500mW/cm<sup>2</sup>

Maße: 288 x 182 mm






# ReOrder Information













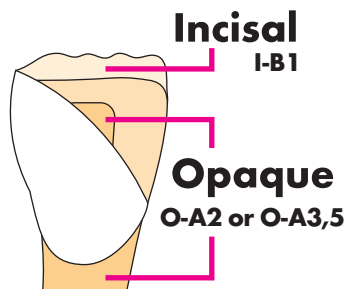

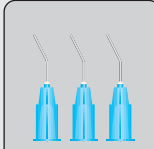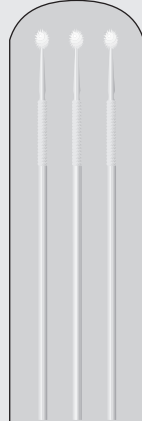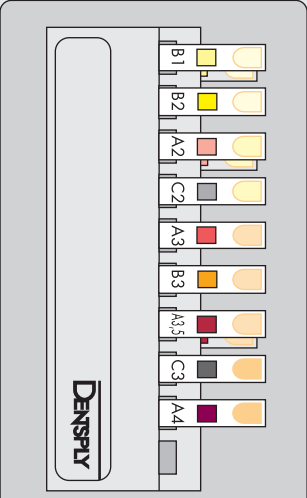

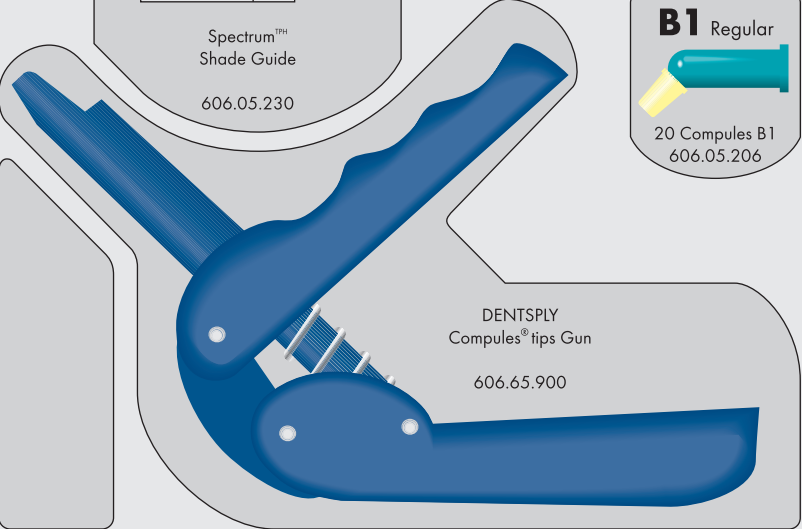

## Spectrum<sup>®</sup>TPH<sup>®</sup>3

Universal microhybrid composite restorative

### Additional Shades

- A1 ReOrder-Nr. 606.05.201
- C4 ReOrder-Nr. 606.05.213
- D3 ReOrder-Nr. 606.05.215



Etch	Bond	Restorative			
 <p>25 Applicator Needles 606.15.204</p>	 <p>100 DENTSPLY Applicator Tips 606.67.198</p>	 <p>Spectrum<sup>™</sup> Shade Guide 606.05.230</p>			
 <p>2 Syringes DeTrey<sup>®</sup> Conditioner 36 3ml each + 25 Applicator Needles 606.15.208</p>	 <p>2 Bottles Prime&amp;Bond<sup>®</sup>NT 4,5ml each 606.67.210</p>	 <p>DENTSPLY Compules<sup>®</sup> tips Gun 606.65.900</p>			
		<p><b>A2 Regular</b></p>  <p>20 Compules A2 606.05.202</p>	<p><b>A3 Regular</b></p>  <p>20 Compules A3 606.05.203</p>	<p><b>A3,5 Regular</b></p>  <p>20 Compules A3,5 606.05.204</p>	<p><b>A4 Regular</b></p>  <p>10 Compules A4 606.05.205</p>
		<p><b>Opaque A2</b></p>  <p>10 Compules O-A2 606.05.220</p>		<p><b>Opaque A3,5</b></p>  <p>10 Compules O-A3,5 606.05.221</p>	
		<p><b>Incisal B1</b></p>  <p>10 Compules I-B1 606.05.222</p>	<p><b>B2 Regular</b></p>  <p>20 Compules B2 606.05.207</p>	<p><b>B3 Regular</b></p>  <p>20 Compules B3 606.05.208</p>	
		<p><b>B1 Regular</b></p>  <p>20 Compules B1 606.05.206</p>			
				<p><b>C2 Regular</b></p>  <p>20 Compules C2 606.05.211</p>	<p><b>C3 Regular</b></p>  <p>20 Compules C3 606.05.212</p>

